KEEP OUT OF THE REACH OF CHILDREN.

Precautions:

If product is damaged or seal opened, sterility is not guaranteed.

Please dispose of the individual sachet after the application because it is no longer sterile and must therefore no longer be used.

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Content:
30 x sachets, 2.7 g each

Returns

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

General Terms and Conditions

The sale, delivery and return of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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EN

Instructions for use for Sterile Lubricating Gel REF 677*

*OptiLube™ Sterile Lubricating Gel: REF 1123

Instructions for use for Sterile Lubricating Gel REF 677 for use with tracheostomy tubes, e.g. TRACOE® twist / twist plus, TRACOE® vario and TRACOE® mini, TRACOE® comfort and TRACOE® larynx

Indication

REF 677 is a lubricating gel for medical applications. It is used in order to carefully and hygienically place tracheostomy tubes in the trachea through an existing stoma.

Contraindications

Sensitivity to Glycerine.

Application

The lubricating gel makes the insertion of tracheostomy tubes easier for patients. Before placing the tube, tear off the corner of a sealed sachet and apply a thin coating of the lubricating gel to the bottom part of the tube including the protruding part of the obturator and the cuff. A pea-sized amount from the individual sachet should be sufficient for each application.

A sealed sachet is intended for one application for one and the same patient.

SIDE EFFECTS AND SPECIAL PRECAUTIONS

Warnings:

After the tracheostomy tube with an adjustable flange has been inserted, the flange position must be checked at regular intervals. In rare cases, the flange may dislocate after the use of lubricating agents.

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