TRACOE tube clean Reinigungsdose

TRACOE tube clean Cleaning Tub

REF 934

DE	Gebrauchsanweisung
EN	Instructions for Use
FR	Instructions d'utilisation
IT	Istruzioni d'uso
ES	Instrucciones de uso

**DE** / Gebrauchsanweisung beachten • **EN** / Please read Instructions for Use • **FR** / Respecter le mode d'emploi • **IT** / Consultare le istruzioni per l'uso • **ES** / Sigla las instrucciones de uso •

**DE** / Medizinprodukt • **EN** / Medical Device • **FR** / Dispositif médical • **IT** / Dispositivo medico • **ES** / Dispositivo médico •

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# Instructions for Use for TRACOE tube clean Cleaning Tub

TRACOE tube clean Cleaning Tub with an inner mesh basket is made of polypropylene and has a capacity of 1000 ml. It is indicated to be used with cleaning solution to clean tracheostomy tubes and placeholders.

## 1. Functional Description

### Caution:

- When the product is used together with other medical devices, follow the respective instructions for use.
- Only one tube, i. e. one outer cannula together with the corresponding inner cannula(s) must be placed next to each other in the mesh basket at the same time to avoid a mix-up when reassembling the tube afterwards.

1. Prepare the cleaning solution (e. g. TRACOE tube clean Cleaning Powder (REF 932) or TRACOE tube clean Concentrated Cleaning Liquid (REF 933)) according to their instructions for use.

2. Use the mesh basket to place the product in the cleaning solution.
3. Cleaning steps for medical devices are described in their respective instructions for use.

## 2. Care and Cleaning

The product is for single patient use only.

If necessary, rinse the tub again with fresh cleaning solution.

Rinse the tub with potable water and dry it off.

The cleaning tub including the mesh basket can be autoclaved using a sterilization process with saturated steam (parameters: one cycle, 20 minutes, 121 °C, 1 bar).

**Caution:** The printing is not dishwasher safe.

## 3. Disposal

Used products are to be disposed of in accordance with valid national regulations, waste management plans, or clinical procedures governing biohazardous waste materials.

The cleaned tub can be disposed of as recyclable waste according to the local regulations.

#### 4. Returns and Complaints

Returned products, that have been used, will only be accepted, if TRACOE has agreed to the return and a completed decontamination certificate and complaint report is enclosed. These forms are available either directly from TRACOE medical, or via the website [www.tracoe.com](http://www.tracoe.com).

If the device is involved in a reportable incident, as defined in local medical device legislation, please contact TRACOE medical ([complaints@tracoe.com](mailto:complaints@tracoe.com)), and the appropriate regulatory body in the country of use.

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TRACOE medical GmbH