TRACOE medical GmbH

Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany

Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

Instructions for Use for TRACOE® *larynx* grid and stoma buttons

TRACOE® grid buttons REF 601, REF 602, REF 603

TRACOE® stoma buttons REF 611, REF 612, REF 613

and for the Accessory TRACOE® stopper REF 621

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Instructions for Use for TRACOE® *larynx* grid and stoma buttons

TRACOE® grid buttons REF 601, REF 602, REF 603
TRACOE® stoma buttons REF 611, REF 612, REF 613
and for the Accessory TRACOE® stopper REF 621

1. General Information

CAUTION: These instructions provide information for physicians, nursing staff and patients. Please read these instructions for use carefully prior to product usage. These instructions relate exclusively to the above products, which are described in the following.

Sterilization: Products are sterilized with ethylene oxide. Sterility can only be guaranteed when the package is completely closed and undamaged. You may resterilize the buttons, however, it should not be necessary in most cases.

Single-Patient Product: The TRACOE® grid and stoma buttons as well as the accessories are medical products intended for only one patient. Multiple use on the same patient is allowed. We strongly recommend that a spare button is kept on hand.

Duration of Use: If cared for properly, TRACOE® grid and stoma buttons can be used for several weeks. Buttons showing even the slightest damage must be replaced immediately.

Storage: The products must be stored in a cool and dry location. Temporary storage for ≤ 30 days at $\leq +50$ °C is possible. Protect the product from sunlight and extreme cold.

CAUTION: United States Federal Law restricts these devices to sale by or on the order of a physician.

Composition/Disposal: The flexible TRACOE® grid and stoma buttons, manufactured of high quality silicone in eight different sizes, are supplied in a standard version as well as in a short and a long version (please see table for dimensions). The silicone used for manufacturing is internationally approved for medical applications. All products are free of latex. Follow applicable national and local guidelines when disposing products.

2. Product Description

2.1 General Properties

TRACOE® grid and stoma buttons feature a specially designed shape and a high level of softness and flexibility. Therefore, they are easy to insert and adapt well to the respective anatomical contours. The bulge at the end of the tube ensures sufficient fixation within the tracheostoma. The silicone used for manufacturing shows excellent tolerability. Combined with high material flexibility this minimizes the risk of irritation or trauma of the trachea as well as potential development of granulation or scarring tissue at the tracheostoma site.

2.2 TRACOE® grid button (REF 601 to REF 603)

The TRACOE® grid button features two lateral eyes (see figure) for fixation using tracheostomy neck straps (e.g. REF 903-F). The TRACOE® grid button is supplied with a protective grid consisting of a concave stainless steel grid (mesh size 0.8 mm) held in a round blue plastic ring that fits exactly into the neck plate of the TRACOE buttons. The protective mesh prevents the aspiration of foreign bodies (>0.8 mm).

Contents: Each box contains a sterile TRACOE® grid button including protective grid, neck strap (REF 903-F), instructions for use, and a patient information card.

2.3 TRACOE® stoma button (REF 611 to REF 613)

The TRACOE® stoma button is supplied without protective grid and without lateral eyes.

Contents: Each box contains a sterile TRACOE® stoma button, instructions for use, and a patient information card.

Size	Order no.	ID (mm)	OD (mm)	short:	standard:	long:
				REF 601 / REF 611	REF 602 / REF 612	REF 603 / REF 613
				Length (mm)	Length (mm)	Length (mm)
06	-06	6	9	16	22	30
07	-07	7	10	16	22	30
08	-08	8	11	16	22	30
09	-09	9	12	16	22	30
10	-10	10	13	16	22	30
11	-11	11	14	16	22	30
12	-12	12	15	16	22	30

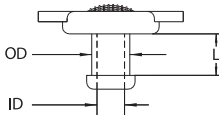
Order number:

REF + size (e.g. REF 601-06)

ID = inner diameter

OD = outer diameter

L = length

2.4 Accessory

Stopper (REF 621)

The stopper consists of a round plate that fits exactly into the neck plate of TRACOE® buttons. For ease of handling, the plate has a central pin on one side. The stopper may be used particularly during weaning for temporary closure of a TRACOE® grid or stoma button if the upper airways are clear and breathing is unobstructed.

Contents: 5 units, supplied in sterile package

CAUTION: Do not use with laryngectomized patients!

Other Accessories

TRACOE medical GmbH also supplies a number of accessories such as a variety of neck straps, various compresses, spacers, shower guards, protective bibs, scarves and roll-necks.

For details, see the general TRACOE Full Line Catalogue.

3. Indications

The TRACOE® grid and stoma button were developed to stabilise the tracheostoma of patients who no longer require a tracheal tube.

4. Instructions for Use

4.1 Note

CAUTION: It is an essential requirement, that the TRACOE® grid and stoma buttons matches with the tracheostoma diameter and length as exactly as possible.

4.2 Insertion and Removal of TRACOE® grid and stoma buttons

TRACOE® grid and stoma buttons can be easily inserted into the tracheostoma. Please fold the stoma button in half between the thumb and index finger and insert the narrow end into the tracheostoma. Release after insertion to allow the button tube to unfold. Push the button carefully into the stoma to complete insertion. The bulge at the end ensures the stoma button is retained within the tracheostoma. If the button is loose please choose a larger size or a TRACOE® grid button which can be fixed by using a neck strap (e.g. REF 903-F) through the lateral eyes. Remove the TRACOE® grid or stoma button by carefully pulling it out of the tracheostoma.

4.3 Insertion and Removal of the Protective Grid

Insert the protective grid of the TRACOE® grid button before inserting it into the tracheostoma. Pull apart the outer ring and place the protective grid into it. Removal of the protective grid can be easily handled by pushing the outer ring of the TRACOE® grid button at both eyes in direction to the tube. Attaching a thread noose at the outer grid-ring prior to insertion makes it even easier to remove the grid by simply pulling the thread. While removing the protective grid, the TRACOE® grid button must be held tight at both eyes or the outer ring, respectively. The protective grid can be inserted and removed in the same way using the TRACOE® stoma button.

4.4 Insertion and Removal of the Stopper (REF 621)

The stopper should preferably be fitted in a TRACOE® grid button or stoma button before the grid or stoma button is introduced into the tracheostoma. This is done by pushing the outer ring of the button apart and inserting the stopper into it. The easiest way of removing the stopper is by holding the outer ring on opposite sides of the button between the fingertips of one hand and pushing it towards the tube, while using two fingers of the other hand to grasp the pin of the stopper and remove it.

Caution: Before inserting a stopper, care must be taken to ensure in all cases that breathing through the larynx, throat, mouth and nose is unobstructed. If there are signs of breathlessness, the stopper must be removed immediately. The stopper must on no account be used in patients following laryngectomy.

5. Cleaning and Disinfection

5.1 Cleaning

Principally, TRACOE® grid and stoma buttons, protective grid, and stopper can be cleaned using commercially available, mild detergents. TRACOE medical recommends to use the fully equipped TRACOE® tube clean cleaning kits (REF 930-A or REF 930-B with cleaning powder and REF 931-A or REF 931-B with concentrated cleaning liquid, respectively) for cleaning TRACOE® grid and stoma buttons. These sets contain cleaning tub, cleaning detergent, cleaning swabs and a cleaning brush. Please follow the respective instructions for use.

CAUTION: The products must never be cleaned with household detergents, denture cleansers or concentrated alcohol as these may have a lasting adverse effect on the function of the product.

5.2 Disinfection

Because the upper respiratory tract is never without germs – even in healthy individuals – we do not recommend the use of disinfection. If, however, disinfection appears necessary, TRACOE medical can provide, upon request, a list of tested disinfectants. This list can also be accessed on the website of TRACOE medical GmbH (www.tracoe.de).

After that, rinse the disinfected TRACOE grid or stoma button thoroughly using distilled water and finally dry it.

CAUTION: Under no circumstances use disinfectants that release chlorine or substances that contain strong alkalis or phenol derivatives.

6. Patient Information Card

You will find the information card enclosed with each TRACOE® grid and stoma button. It contains all product details (REF no., size, length, lot no.) and should be stored separately. The card facilitates reordering and must be attached when sending product complaints.

7. Returns

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

8. General Terms and Conditions

Sales and deliveries of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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